SolutionX Core

PATIENTS

EN	Instructions for use.....	2
SE	Bruksanvisning	6
DE	Gebrauchsanweisung	10

SolutionX Core

PATIENT INSTRUCTIONS FOR USE (EN)

Product

SolutionX Core

Product type

Ankle-foot orthosis

Language

English

Manufacturer

O&P Solutions AB

Important advice and tips

This brochure explains how to use and care for your new orthosis.

SolutionX Core is intended to maintain the foot in a neutral position and prevent drop foot. This supports a more natural gait and reduces the risk of tripping.

Its flexibility and design help provide a more stable and secure gait.

We hope that the function, discreet design and low weight of SolutionX Core will make it a comfortable part of your daily life.

X

High-impact activities

High-impact movements can damage the material.

✓

Standing up

Place both feet firmly on the floor and use the armrests for support.


✓

Using stairs

Place the entire foot securely on each step.

✓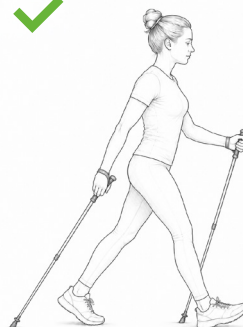

Stability

Walking poles can improve stability and confidence.

Putting on SolutionX Core

- Place the footplate in a suitable shoe, underneath the shoe's insole or an orthotic insert.
- Push the orthosis fully back against the heel counter of the shoe.
- Open the shoe wide and insert the foot. The calf cuff should rest comfortably against the calf.
- Fasten the padded calf strap.
- Secure the shoe. Check that the orthosis is correctly positioned and does not rub.

Removing SolutionX Core

- Open the hook-and-loop strap.
- Loosen the laces or open the shoe, then remove the foot.
- You may leave the orthosis in the shoe.

Remember

- Stable shoes work best.
- Wear cotton knee-high socks.
- Always place the footplate beneath an insert or the shoe's insole.
- Inspect your AFO every day so that wear is detected early.
- Contact your healthcare professional if the orthosis rubs or feels wrong.

For your use only

This device has been fitted and adjusted specifically for your needs and must only be used by you.

Do not adjust it yourself

All adaptations and adjustments must be carried out by a healthcare professional. Do not bend, heat or adjust the device yourself.

The right footwear makes a difference

SolutionX Core is made from a thin, flexible material and therefore fits most conventional shoes. For optimal function, wear a stable shoe with a heel height of approximately 1–2 cm. If you are unsure, ask your healthcare professional.

Do not wear slippers or sandals, as they do not provide sufficient support for the orthosis.

Check the shoe before use. Sand and grit can damage the orthosis.

Avoid high-impact activities

SolutionX Core is designed to support an active daily life for people with drop foot, but it is not suitable for jumping, running or other high-impact physical activities.

Choose suitable socks

For optimal comfort, we recommend cotton knee-high socks. They help reduce rubbing and protect the leg.

Protect the foot and lower leg

The SolutionX Core footplate must always be placed inside the shoe, underneath an orthotic insert or the shoe's insole. If you have difficulty removing the insole, contact your healthcare professional.

The inside of the calf cuff where the hook-and-loop strap is attached is softly padded to protect the leg from pressure and rubbing. If the padding wears out, it can be replaced. Your healthcare professional can assist you.

Caring for your AFO

Clean the orthosis, padding and straps with mild soap and water. Allow all parts to air-dry. Do not dry them in direct sunlight.

Driving

We do not recommend driving while wearing the orthosis because an AFO restricts foot movement and may therefore affect your reaction time. Always consult your healthcare professional.

If something does not feel right

If the orthosis feels uncomfortable, causes rubbing, or you experience pain, numbness or swelling, contact your healthcare professional as soon as possible.

Damage and wear

Inspect the orthosis every day. If you notice cracks or other damage, stop using it and contact your healthcare professional. The orthosis must be replaced if damaged.

If a serious injury occurs

If you sustain a serious injury while using the device, contact your healthcare professional immediately. You may also contact the manufacturer with questions or to report an incident.

Expected functional lifetime

The expected functional lifetime of SolutionX Core is 3 years under normal use. Actual service life depends on frequency of use, the patient's activity level and correct maintenance. Replace the device immediately if cracks, deformation or loss of function occur.

Storage and transport

Store the device in a clean, dry place, protected from direct sunlight, heat sources, moisture and chemicals. After cleaning, allow the orthosis to air-dry at room temperature. Do not place heavy objects on the orthosis or subject it to other mechanical loads, as this may cause deformation. During transport, package and handle the orthosis so that mechanical damage is prevented.

Disposal

The device contains no hazardous substances. At the end of its service life, dispose of it in accordance with the local and national regulations applicable in the country of use. This applies to patients, clinics and distributors.

Garanti

O&P Solutions AB provides a 12-month warranty covering manufacturing defects only. The warranty applies provided that the orthosis has been fitted and used in accordance with these instructions for use. It does not cover damage caused by misuse, unauthorised modification or overloading. For warranty claims, contact O&P Solutions AB and provide the product lot/batch number.

Serious incidents

Any serious incident that has occurred in relation to the device must be reported to O&P Solutions AB and to the competent authority of the Member State in which the user and/or patient is established.

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Tillverkare: O&P Solutions AB