SolutionX Core

PROFESSIONALS

EN	Instructions for use.....	2
SE	Bruksanvisning	8
DE	Gebrauchsanweisung	14

SolutionX Core

PROFESSIONAL

INSTRUCTIONS FOR USE (EN)

Product

SolutionX Core

Product type


Ankle-foot orthosis

Language

English

Manufacturer

O&P Solutions AB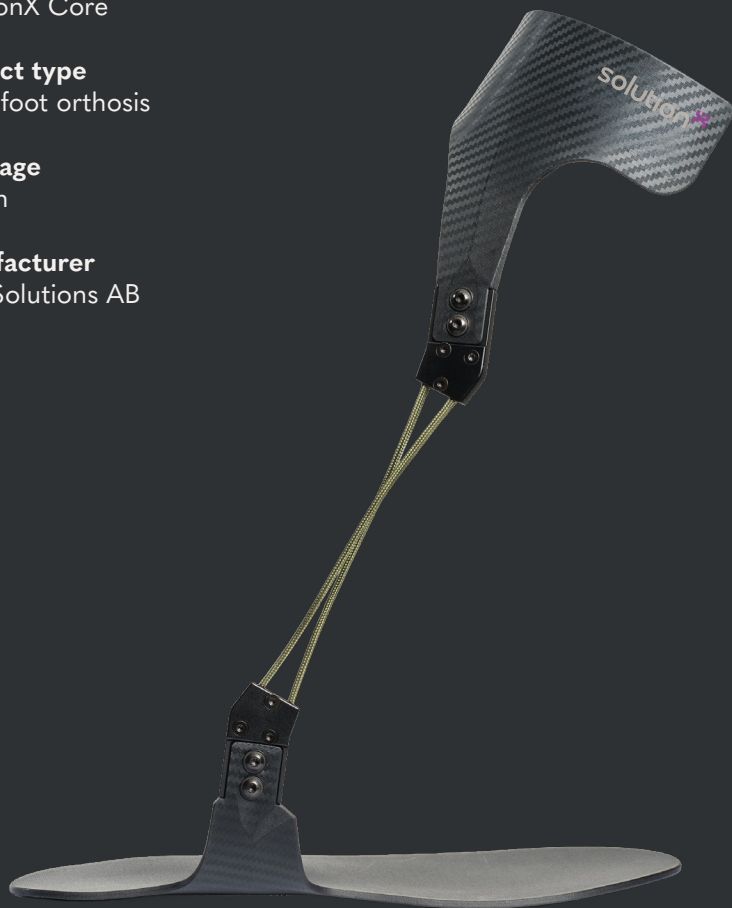

Product identification

REF	Product description	Side	Size	Shoe size
SX-CR-L-L	SolutionX Core Large Left	Left	Large	42-47
SX-CR-L-R	SolutionX Core Large Right	Right	Large	42-47
SX-CR-M-L	SolutionX Core Medium Left	Left	Medium	< 42
SX-CR-M-R	SolutionX Core Medium Right	Right	Medium	< 42

Important information

Read the entire instructions for use before fitting or using the device. It is intended for a single patient only. The manufacturer's warranty does not apply if it is used by multiple patients.

Intended use

SolutionX Core is an ankle-foot orthosis for adults with drop foot or impaired dorsiflexion resulting from a neurological condition or injury. It assists dorsiflexion during walking and helps promote a more natural gait. The device must be fitted by a licensed orthotist or another healthcare professional with relevant competence. The patient may then use it in daily life. SolutionX Core may be used bilaterally, with one orthosis on each leg, following individual fitting.

Indications

SolutionX Core is indicated for patients with drop foot due to a neurological condition or injury, or with weakness of the ankle dorsiflexor muscles.

Contraindications

Do not use the device in the presence of:

- open wounds or skin lesions in the area of application
- severe foot deformities that prevent correct fitting
- moderate to severe spasticity in the lower limb
- known hypersensitivity to any of the device materials

Accessories

SolutionX Padding Kit, REF SX-PAD-M and SX-PAD-L, is sold separately. The use of other, unapproved accessories may affect device function and safety.

Warnings

WARNING: The device must be fitted and adjusted only by a licensed orthotist or another healthcare professional with relevant competence.

WARNING: Only the dorsiflexion adjustment by bending the joint as described in these instructions is intended. No other parts may be bent, heated, drilled or modified.

WARNING: Do not use the device if cracks, fractures or deformation are visible. A damaged device may increase the risk of falls or further injury.

WARNING: Discontinue use immediately if pain, numbness, swelling or skin injury occurs.

WARNING: The device is not intended for running, jumping or other high-impact activities. Such use may cause device failure and personal injury.

WARNING: Driving or operating machinery while wearing the device should occur only after consultation with a healthcare professional, as it may affect ankle mobility and reaction time.

Precautions

- Inspect the skin regularly, particularly during the first few weeks, for redness or irritation.
- Always use the device with suitable footwear that provides stable heel support. Sandals and clogs are not suitable.
- Wear a sock or equivalent protective layer under the orthosis to reduce the risk of skin irritation.
- Check the straps, padding and fasteners regularly. Replace worn components promptly.
- Ensure that the device fits correctly and is securely fastened before walking.
- Monitor patients with reduced sensation in the lower limb closely for skin injury.

Assembly, fitting and adjustment

This section is intended for a licensed orthotist or another healthcare professional with relevant competence. SolutionX Core is available in Medium and Large. The footplate may need to be adjusted in length and width for an optimal fit in the shoe.

WARNING: Patients must not attempt to adjust or modify the device themselves.



WARNING: Incorrect adjustment may impair function and safety and lead to cracking, delamination or fracture.

Assembling the AFO



1. Attach the foot section to the side of the joint marked with the lot/batch number and the calf section to the opposite side of the joint.
2. Apply Loctite 222 to the screws and install two screws in each section.
3. Tighten the screws to 1.5 Nm.

Adjusting the footplate

1. Place the shoe insole on top of the footplate.
2. Position the orthosis so that the lateral joint lies behind the base of the fifth metatarsal and in front of the lateral malleolus.
3. Once correctly positioned, use the insole as a template.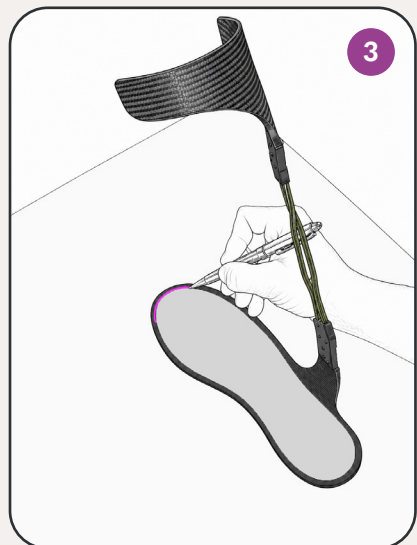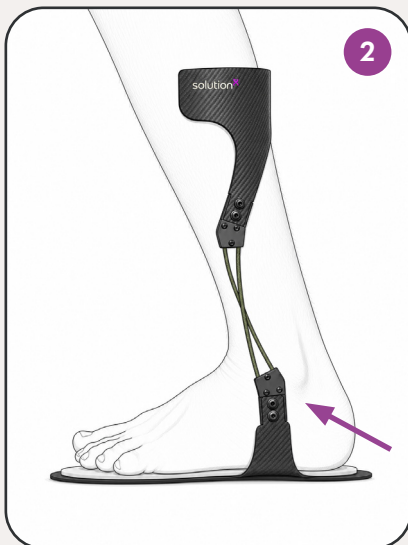

4. Trim the footplate with a suitable grinding machine. Always grind along the length of the footplate, never across it. To make insertion into the shoe easier, approximately 10 mm may be removed from the front of the footplate. This is an approximate recommendation, not a maximum permitted amount. Base the adjustment on the shoe, foot and desired fit.
5. After adjustment, smooth the edges with fine sandpaper or wet sanding so that no sharp edges remain.
6. Ensure that the orthosis is fully pushed back against the heel counter of the shoe.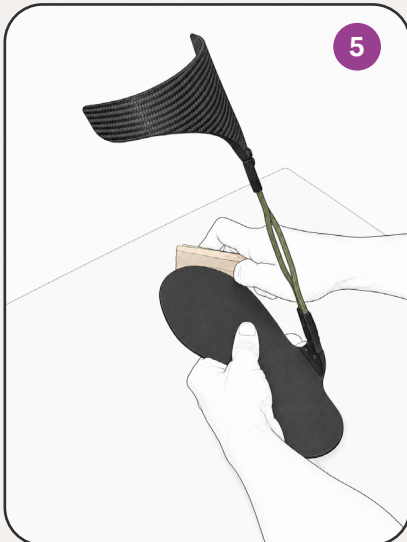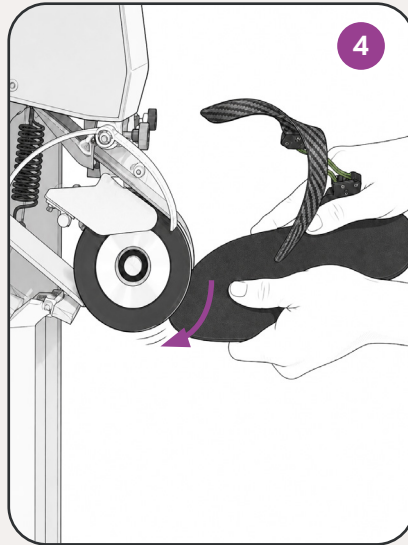

Checking rotation in the shoe

Depending on the shape of the shoe, the orthosis may rotate inward or outward. Adjust the footplate carefully so that the joint remains stable and does not twist inside the shoe.

Important: Never grind or adjust the lateral strut where it attaches to the footplate. This may impair function and weaken the orthosis, increasing the risk of cracks or fracture.


Dorsiflexion adjustment procedure

Only the following bending of the joint is an intended adjustment:

1. Hold only the upper and lower aluminium mounts directly adjacent to the crossed leaf springs.
2. Bend gently and only in the intended dorsiflexion direction, with the toes pointing upward.
3. Release the load. Repeat only if necessary until the desired permanent dorsiflexion adjustment is achieved.
4. After adjustment, check that the joint moves freely and that SolutionX Core has no visible damage.

Note:

- Do not hold the distal polymer components below the joint.
- Do not hold the proximal polymer components above the joint.
- Do not apply torsion or medial or lateral bending.
- Do not bend, heat, drill or modify any other parts.

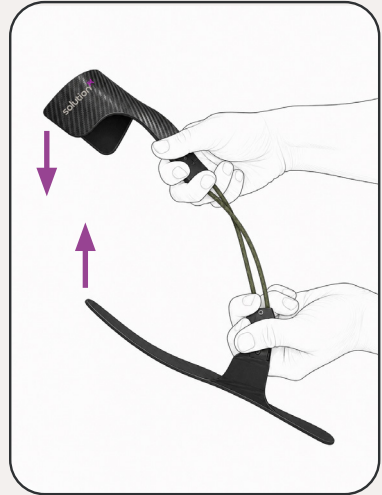Patient instructions

The fitting professional is responsible for ensuring that the patient receives and understands the following instructions before use.

Donning

1. Place the orthosis in a suitable shoe with the footplate beneath the shoe insole.
2. Ensure that the orthosis is fully pushed back against the heel counter.
3. Open the shoe wide and insert the foot. The calf section should rest against the calf.
4. Place the padded side of the strap against the leg and fasten the hook-and-loop closure.
5. Lace or close the shoe securely.

Doffing

Release the calf strap. One side may remain attached to make the next donning easier. Open the shoe and remove the foot while leaving the orthosis in the shoe.

Protection against chafing

Always wear a sock that extends above the orthosis. Inspect the skin regularly for signs of chafing or pressure.

Daily inspection

Inspect the orthosis daily for cracks and unusual wear. If pain, swelling, numbness or chafing occurs, discontinue use and contact the healthcare professional.

Cleaning and maintenance

The same cleaning instructions apply to the orthosis, padding and straps:

- Clean with mild soap and water.
- Do not machine-wash or dry-clean.
- Do not use solvents.
- Allow to air-dry at room temperature, away from direct heat and sunlight.

Inspect the device daily and replace worn components promptly. If a defect is found, discontinue use and contact the clinic.

Expected functional lifetime

The expected functional lifetime of SolutionX Core is 3 years under normal use. Actual service life depends on frequency of use, the patient's activity level and proper maintenance. Replace the device immediately if cracks, deformation or loss of function occurs.

Storage and transport

Store the device in a clean, dry place, protected from direct sunlight, heat sources, moisture and chemicals. After cleaning, allow the orthosis to air-dry at room temperature. Do not subject it to mechanical loads, such as placing heavy objects on top of it, as this may cause deformation. During transport, package and handle the orthosis to prevent mechanical damage.

Disposal

The device contains no hazardous substances. At the end of its service life, dispose of it in accordance with the local and national regulations applicable in the country of use. This applies to patients, clinics and distributors.

Warranty

O&P Solutions AB provides a 12-month warranty covering manufacturing defects only. The warranty applies provided that the orthosis has been fitted and used in accordance with these instructions for use. It does not cover damage caused by misuse, unauthorised modification or overloading. For warranty claims, contact O&P Solutions AB and provide the product lot/batch number.

Serious incidents

Any serious incident that has occurred in relation to the device must be reported to O&P Solutions AB and to the competent authority of the Member State in which the user and/or patient is established.

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Hersteller: O&P Solutions AB