SolutionX Joint

PROFESSIONALS

EN	Instructions for use.....	2
SE	Bruksanvisning	9
DE	Gebrauchsanweisung	16

SolutionX Joint

INSTRUCTIONS FOR USE FOR PROFESSIONAL (EN)

Product

SolutionX Joint

Product type


Joint component for an
ankle-foot orthosis

Language

English

Manufacturer

O&P Solutions AB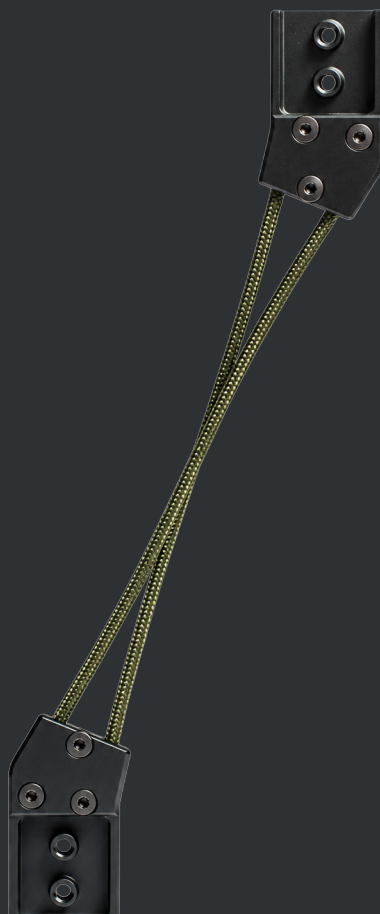

Product identification

REF	Product description	Side
SX-JNT-L	SolutionX Joint Left	Left
SX-JNT-R	SolutionX Joint Right	Right

Important information

Read these instructions in full before assembly, fitting or use. SolutionX Joint must be integrated into a compatible SolutionX orthosis and individually fitted by a licensed orthotist or another healthcare professional with relevant expertise.

Intended purpose

SolutionX Joint is a joint component for ankle-foot orthoses intended for adults with foot drop or impaired dorsiflexion resulting from a neurological condition or injury. The device supports dorsiflexion during gait and helps promote a more natural gait pattern. It is intended to be integrated and fitted in a clinical setting by a licensed orthotist or another healthcare professional with relevant expertise. The completed orthosis is then used by the patient in daily life.

Indications

SolutionX Joint is intended for patients with foot drop resulting from a neurological condition or injury and for patients with weakness of the ankle dorsiflexor muscles.

Contraindications

The device must not be used in the presence of:

- open wounds or skin lesions in the area of contact
- severe foot deformities that prevent correct fitting
- moderate to severe spasticity of the lower limb
- known hypersensitivity to any of the device materials

Varningar

WARNING: The device may only be assembled, fitted and adjusted by a licensed orthotist or another healthcare professional with relevant expertise.

WARNING: Only the dorsiflexion adjustment achieved by bending the joint as described in these instructions is intended. No other part may be bent, heated, drilled or modified.

WARNING: Do not use the device if cracks, fractures or deformations are visible. Use of a damaged device may increase the risk of falls or further injury.

WARNING: Discontinue use immediately if pain, numbness, swelling or skin damage occurs.

WARNING: The device is not intended for running, jumping or other high-impact activities. Such use may result in device failure and personal injury.

WARNING: Driving or operating machinery while wearing the device should only be undertaken after consultation with a healthcare professional, as the device may affect ankle mobility and reaction time.

Precautions

- Inspect the skin regularly, particularly during the first few weeks, for redness or irritation.
- Always use the device with suitable footwear that provides stable heel support. Sandals and clogs are not suitable.
- Wear a sock or equivalent protective layer beneath the orthosis to reduce the risk of skin irritation.
- Regularly inspect the straps, padding and fasteners. Replace worn components immediately.
- Ensure a correct fit and confirm that the device is securely fastened before walking.
- Patients with reduced sensation in the lower limb must be monitored closely for skin damage.

Compatible components

SolutionX Joint may be used with the components listed below. No other accessories are required. Use of unapproved components may affect device function and safety.

REF	Product description
SX-LA-001	SolutionX Lamination Anchor
SX-PAD-M	SolutionX Padding Medium
SX-PAD-L	SolutionX Padding Large
FS-CR-L-L	Foot Section Core Large Left
FS-CR-L-R	Foot Section Core Large Right
FS-CR-M-L	Foot Section Core Medium Left
FS-CR-M-R	Foot Section Core Medium Right
FS-CRF-L-L	Foot Section Core Large Unformed Left
FS-CRF-L-R	Foot Section Core Large Unformed Right
FS-CRF-M-L	Foot Section Core Medium Unformed Left
FS-CRF-M-R	Foot Section Core Medium Unformed Right
FS-APX-L-L	Foot Section Apex Large Left
FS-APX-L-R	Foot Section Apex Large Right
FS-APX-M-L	Foot Section Apex Medium Left
FS-APX-M-R	Foot Section Apex Medium Right
CS-CR-L-L	Calf Section Core Large Left
CS-CR-L-R	Calf Section Core Large Right
CS-CR-M-L	Calf Section Core Medium Left
CS-CR-M-R	Calf Section Core Medium Right
CS-CRF-L-L	Calf Section Core Large Unformed Left
CS-CRF-L-R	Calf Section Core Large Unformed Right
CS-CRF-M-L	Calf Section Core Medium Unformed Left
CS-CRF-M-R	Calf Section Core Medium Unformed Right
CS-APX-L-L	Calf Section Apex Large Left
CS-APX-L-R	Calf Section Apex Large Right
CS-APX-M-L	Calf Section Apex Medium Left
CS-APX-M-R	Calf Section Apex Medium Right

Assembly fitting and adjustment

This section is intended for a licensed orthotist or another healthcare professional with relevant expertise. SolutionX Joint is available in left and right versions.

WARNING: The patient must not attempt to adjust or modify the device.

WARNING: Incorrect adjustment may compromise the function and safety of the orthosis and may result in cracking, delamination or fracture.

Assembly of compatible components



When using a Core or Apex foot section, calf section or lamination anchor, assemble the components to SolutionX Joint using the four screws supplied.

1. Attach the foot section to the side of the joint marked with the lot/batch number and attach the calf section to the opposite side of the joint.
2. Apply Loctite 222 to the screws and install two screws in each section.
3. Tighten the screws to 1.5 Nm.




Footplate adjustment

This section applies when SolutionX Joint is used with a compatible SolutionX foot section.

1. Place the shoe insole on top of the footplate.
2. Position the orthosis so that the lateral joint is behind the base of the fifth metatarsal and anterior to the lateral malleolus.
3. Once the position is correct, use the insole as a template.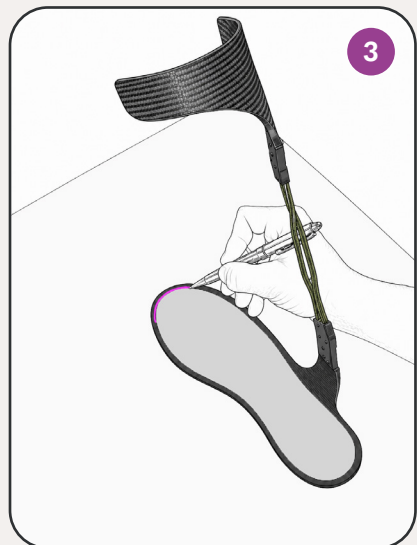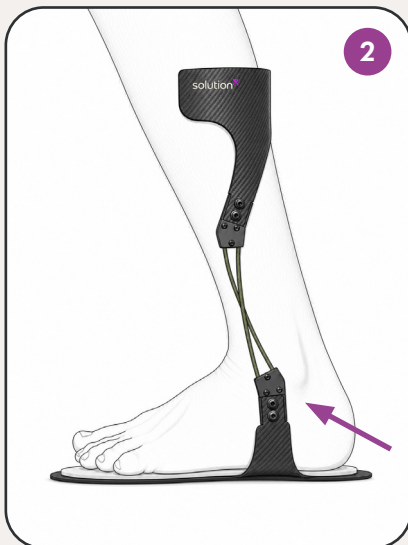

4. Grind the footplate using suitable grinding equipment. Always grind in the longitudinal direction of the foot plate, never across the blade. To facilitate insertion into the shoe, the front of the footplate may be shortened by approximately 10 mm. This measurement is an approximate recommendation, not a maximum permitted limit. The final adjustment must be based on the shoe, the foot and the required fit.
5. After adjustment, smooth the edges with fine abrasive paper or by wet sanding so that no sharp edges remain.
6. Ensure that the orthosis is fully seated against the heel counter of the shoe.

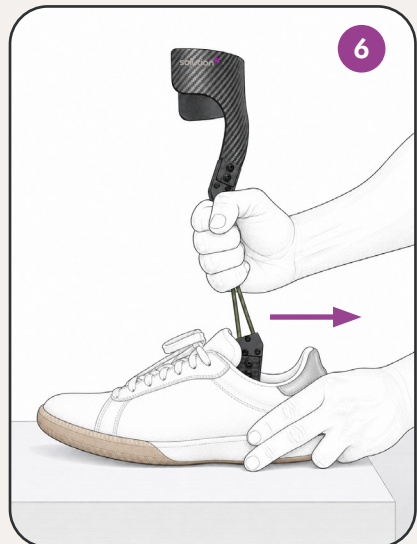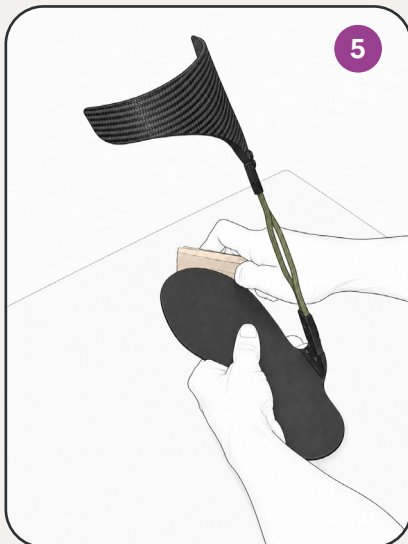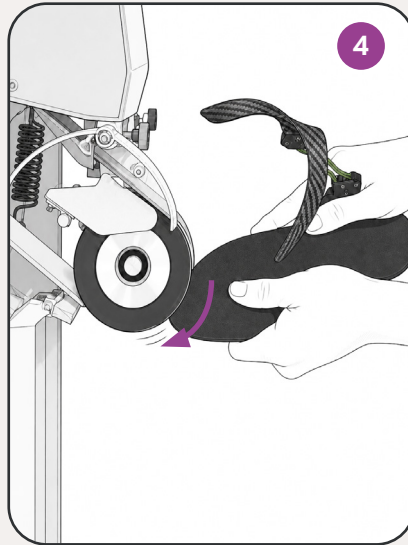Checking rotation in the shoe

Depending on the shape of the shoe, the orthosis may rotate medially or laterally. Adjust the footplate carefully so that the joint remains stable and does not twist inside the shoe.

Important. Never grind or adjust the lateral strut where it is attached to the footplate. Doing so may impair function and weaken the orthosis, increasing the risk of cracks or fracture.


Dorsiflexion adjustment procedure

Only the following bending of the joint is an intended adjustment:

1. Hold only the upper and lower aluminium mounts immediately adjacent to the crossed leaf springs.
2. Bend carefully and only in the intended dorsiflexion direction, with the toes moving upward.
3. Release the load. Repeat only as necessary until the required permanent dorsiflexion adjustment has been achieved.
4. After adjustment, check that the joint moves freely and inspect SolutionX Joint and the completed orthosis for visible damage.

Important:

- Do not hold the distal polymer components below the joint.
- Do not hold the proximal polymer components above the joint.
- Do not apply torsion or bend the joint medially or laterally.
- Do not bend, heat, drill or modify any other part.

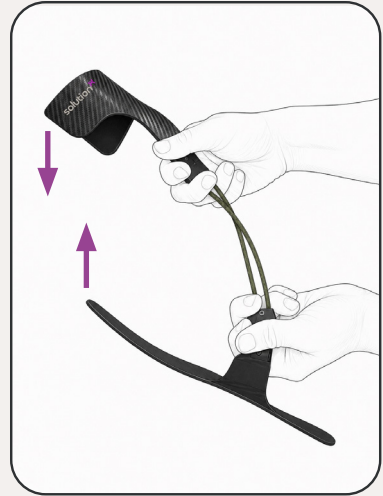Patient instructions

The fitting professional is responsible for ensuring that the patient receives and understands the following instructions before use.

Donning

1. Place the orthosis in a suitable shoe with the footplate beneath the shoe insole.
2. Ensure that the orthosis is fully seated against the heel counter.
3. Open the shoe fully and insert the foot. The calf section must rest against the calf.
4. Position the strap with the padding against the leg and secure the hook-and-loop closure.
5. Lace or close the shoe securely.

Doffing

Release the calf strap. One side may be left attached to make the next donning easier. Open the shoe and remove the foot while leaving the orthosis in the shoe.

Protection against chafing

Always wear a sock that extends above the orthosis. Check the skin regularly for signs of chafing or pressure.

Daily inspection

Inspect the orthosis daily for cracks or abnormal wear. If pain, swelling, numbness or chafing occurs, discontinue use and contact the healthcare provider.

Cleaning and maintenance

The same cleaning instructions apply to the orthosis, padding and straps:

- Clean with mild soap and water.
- Do not machine wash or dry clean.
- Do not use solvents.
- Allow to air dry at room temperature, away from direct heat and sunlight.

Inspect the device daily and replace worn components immediately. If a defect is identified, discontinue use and contact the clinic.

Expected functional lifetime

The expected functional lifetime of SolutionX Joint is 3 years under normal use. Actual service life depends on frequency of use, the patient's activity level and correct maintenance. Replace the device if deformation or loss of function occurs.

Storage and transport

When SolutionX Joint is used with a Core or Apex foot section and calf section, store the completed orthosis in a clean, dry place, protected from direct sunlight, heat sources, moisture and chemicals. After cleaning, allow the orthosis to air dry at room temperature. Do not subject the orthosis to mechanical loads, such as placing heavy objects on top of it, as this may cause deformation. During transport, package and handle the orthosis in a manner that prevents mechanical damage.

Disposal

The device contains no hazardous substances. At the end of its service life, dispose of the device in accordance with the local and national regulations applicable in the country of use. This applies to patients, clinics and distributors.

Warranty

O&P Solutions AB provides a 12-month warranty covering manufacturing defects only. The warranty applies provided that SolutionX Joint has been assembled and the completed orthosis has been fitted and used in accordance with these instructions for use. The warranty does not cover damage resulting from misuse, unauthorised modification or overloading. For warranty claims, contact O&P Solutions AB and provide the product lot/batch number.

Serious incidents

Any serious incident that has occurred in relation to the device must be reported to O&P Solutions AB and to the competent authority of the Member State in which the user and/or patient is established.

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Hersteller: O&P Solutions AB